


OWNER'S MANUAL

Tilt Soft Form Series

With Motion Sensor

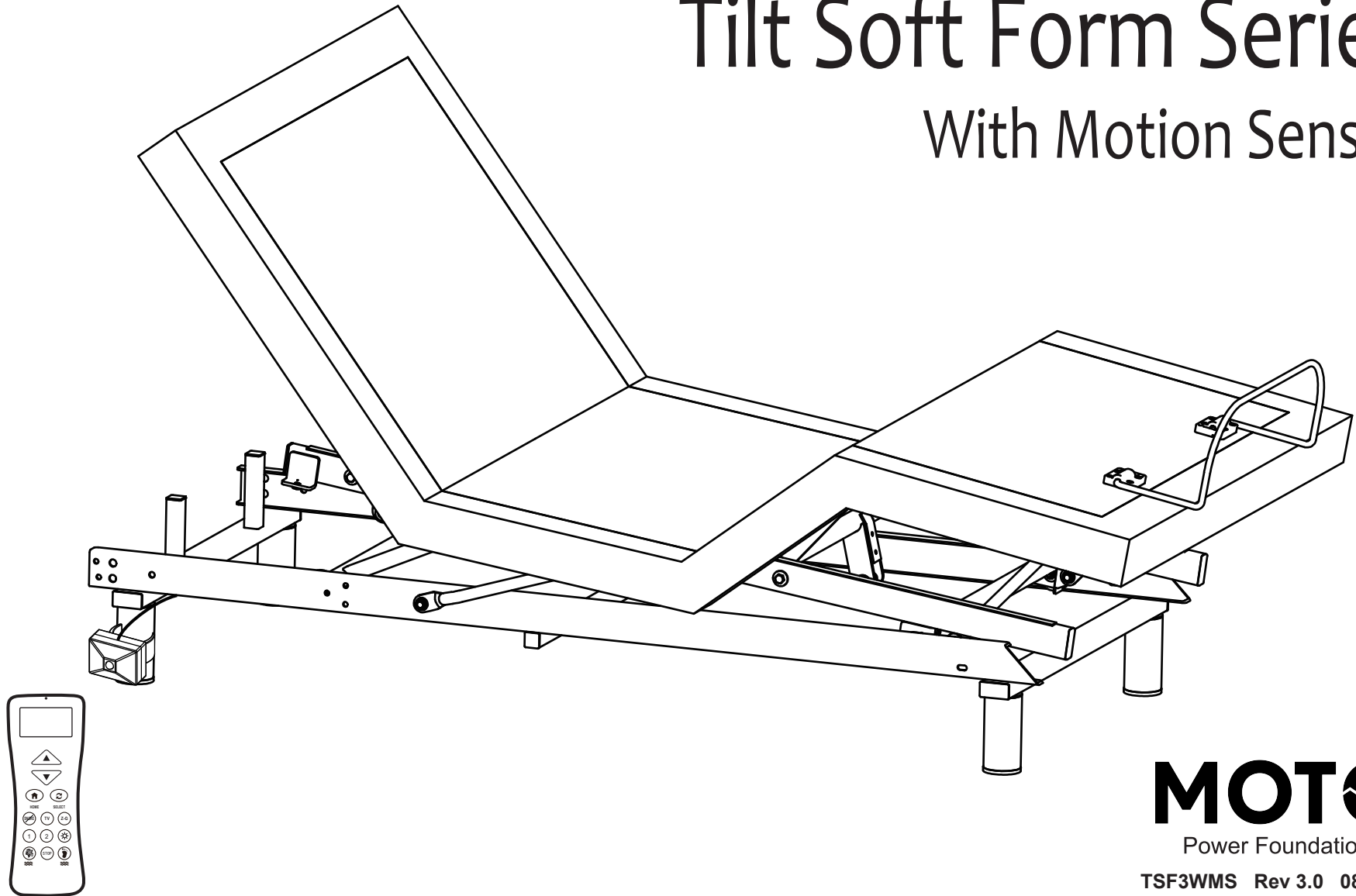


MOTO

Power Foundations

TSF3WMS Rev 3.0 08/2023

Installation Guide

1 Open the box lid.

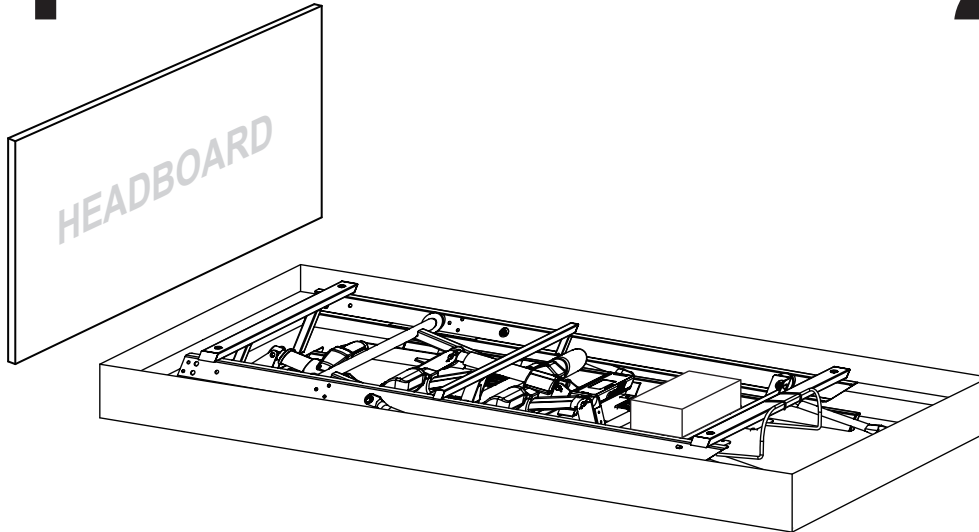


2 Take out all the accessories from the packaging and put them aside. All electronics and components that need to be installed are located in boxes under the foundation or attached to the frame. Before discarding the packing materials, ensure that all the parts are accounted for.

Mattress Retainer Bar




x1

Accessories Box

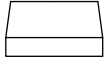


x1


ACCESSORIES BOX:

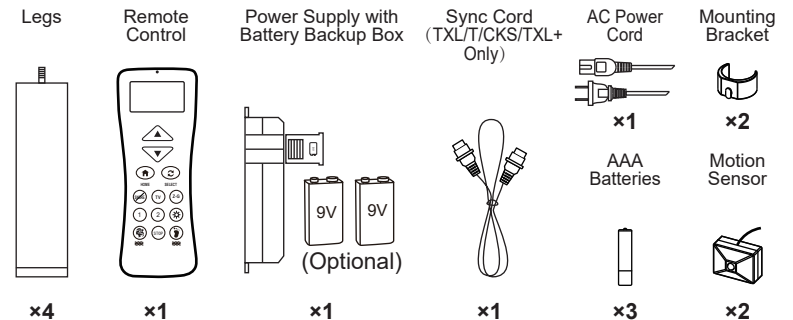

3 Install 4 legs, make sure the legs are tight.

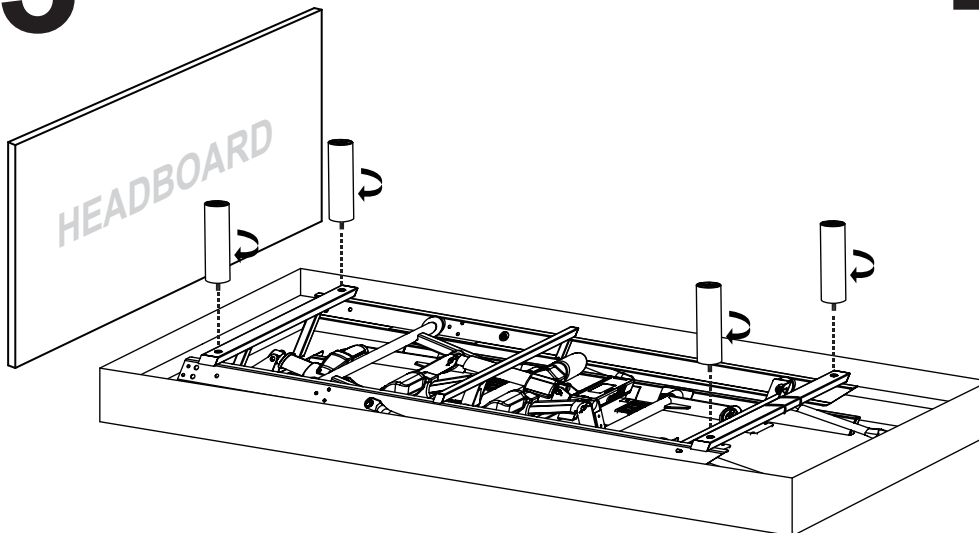


4 Take the bed out of the box, turn over onto 4 legs. Remove the box. Uncoil the DC power cord (connected to the control box) and put it to the side of the bed.

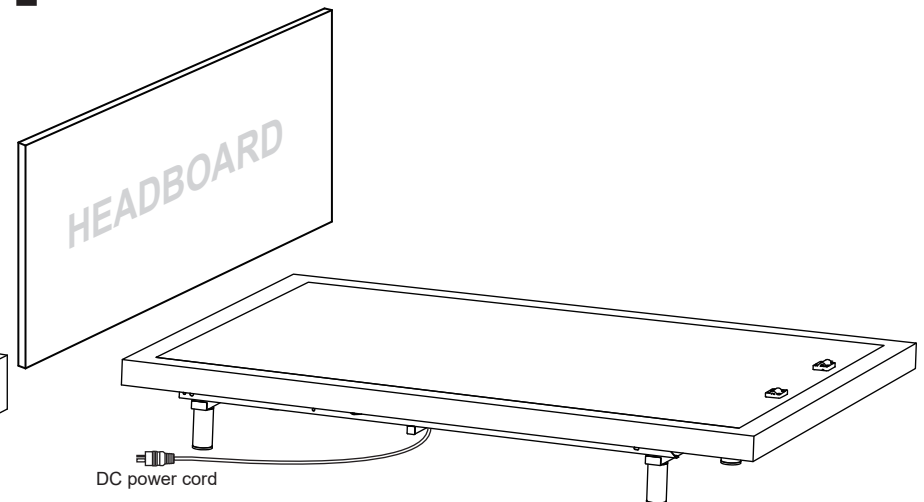


5

Motion sensor installation

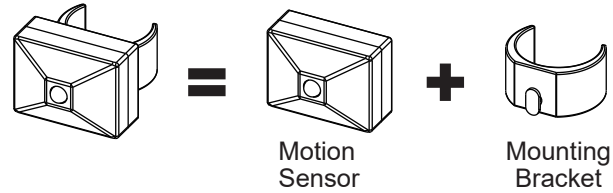


Prepare The Motion Sensor Hardware.


STEP 1: Connect the motion sensor to the mounting bracket.

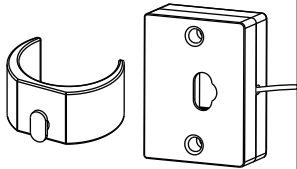

STEP 2: Rotate the mounting bracket 90 degrees clockwise

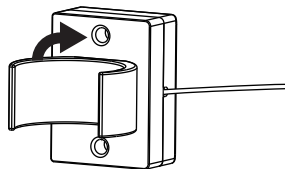


STEP 3:



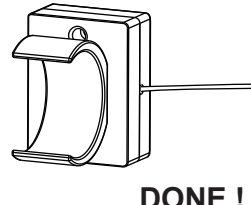


Connect The Motion Sensor Connectors.

STEP 1: Clip the mounting bracket onto the head base legs as illustrated.

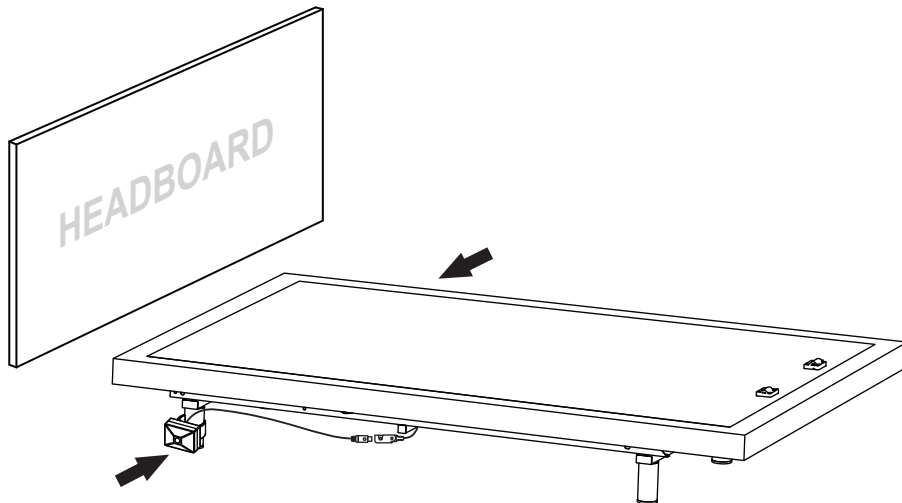


STEP 2: Connected the motion sensor connectors as illustrated. Note there are two connectors, one on each side of the bed.

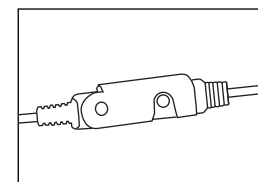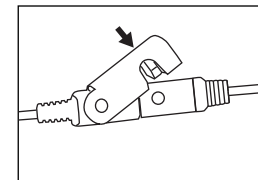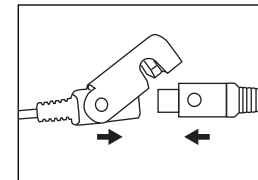


6

Install the mattress retainer bar one side after another.

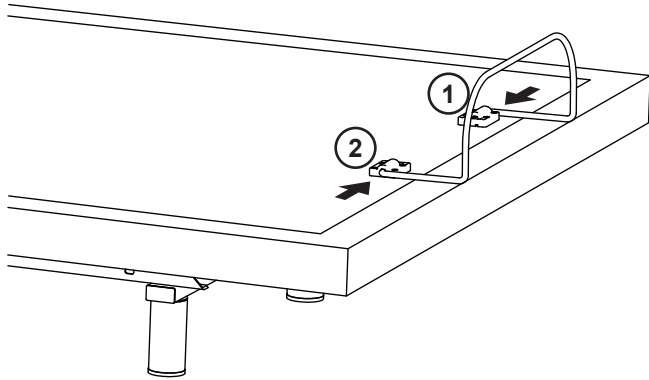


7


Uncoil the AC power cord, connect the DC power cord to the power supply and then connect the power supply to the AC power source.

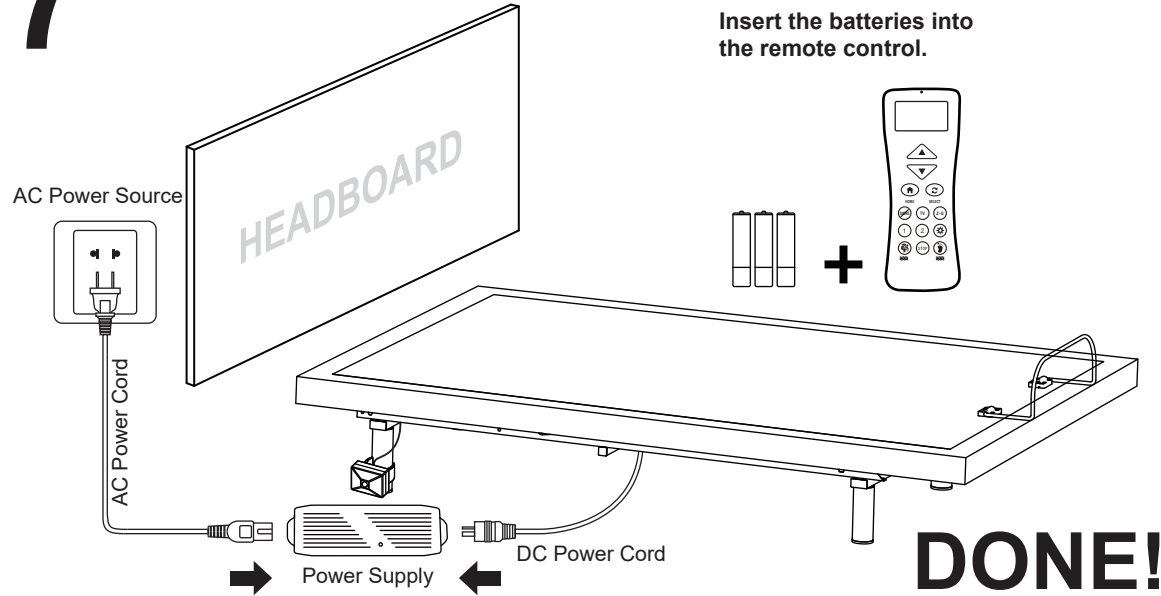

Electronics Quick Reference Guide

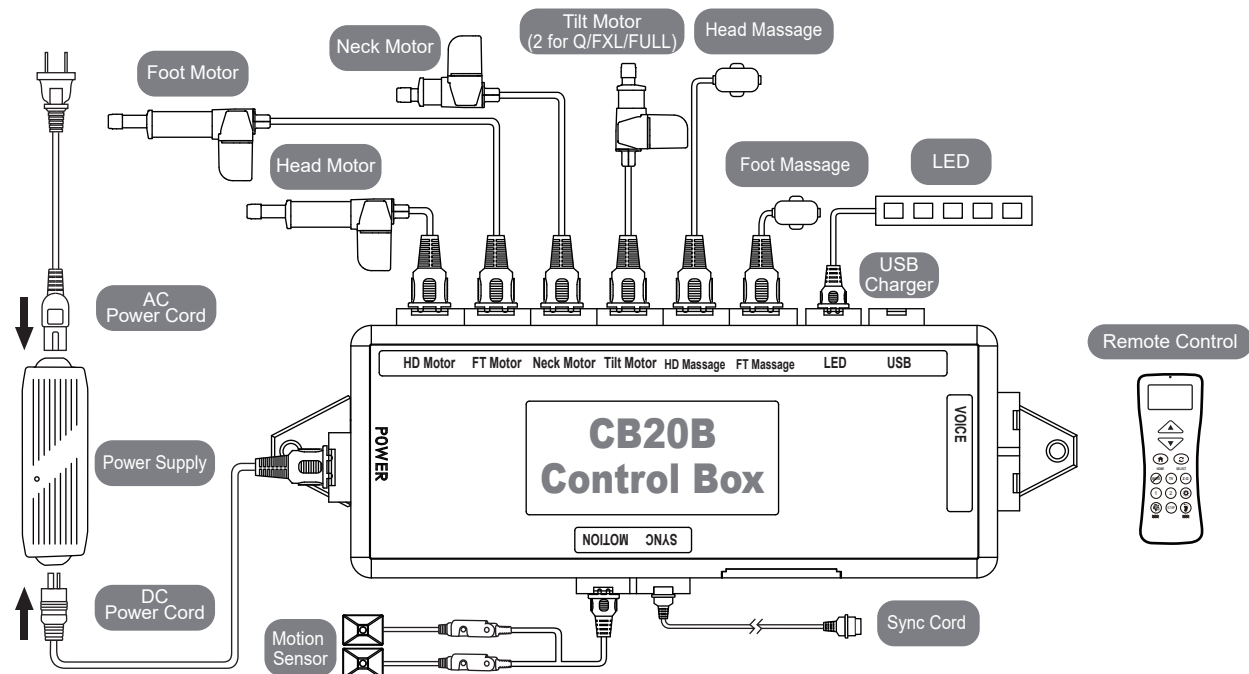

Remote Control Instructions

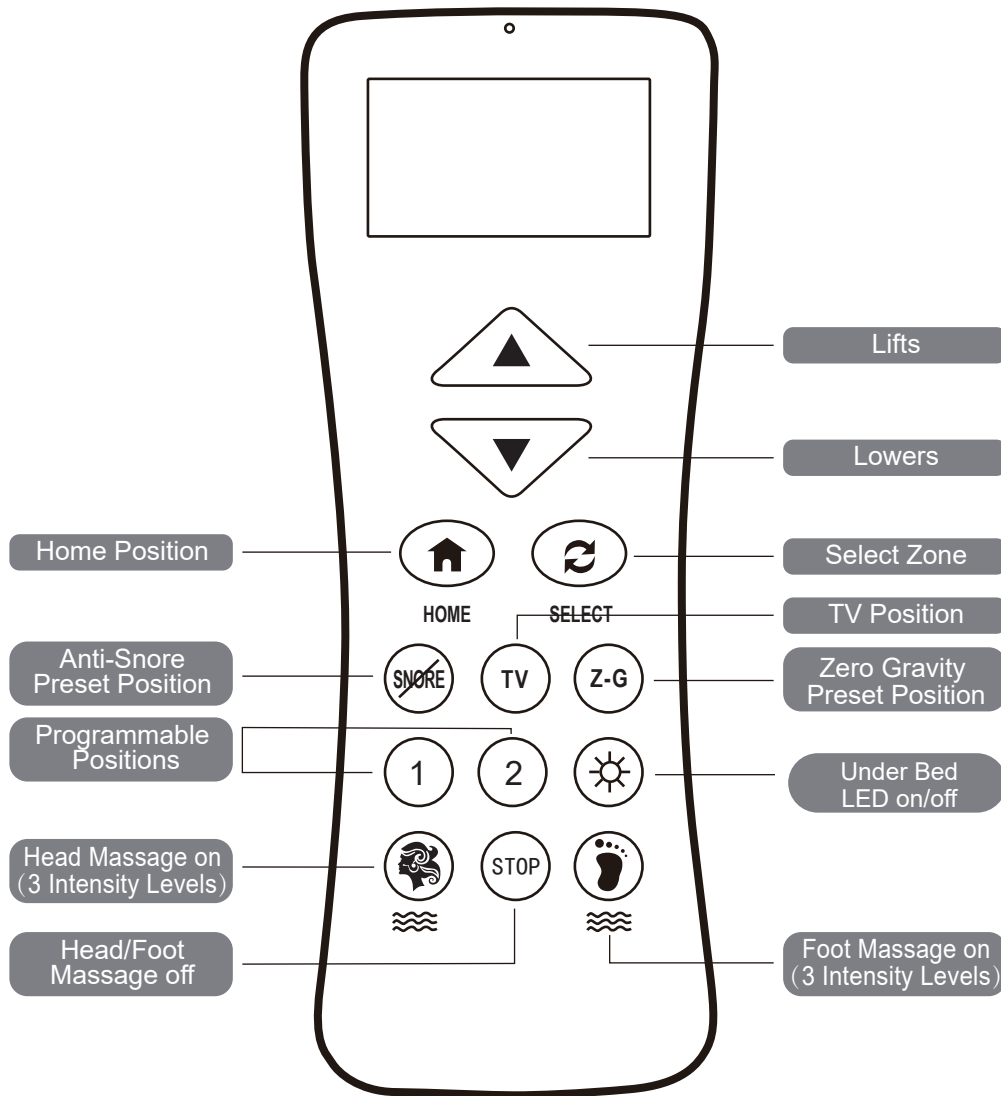

ADJUST

- ▲ This button lifts the head/foot/neck/tilt section of the foundation.
- ▼ This button lowers the head/foot/neck/tilt section of the foundation.

SETTING PROGRAMMABLE POSITION

- 1 One touch programmable positions.
- 2

MASSAGE FEATURES


- Head massage icon: Turn on the head massage. Cycle through 3 massage intensities.
- Foot massage icon: Turn on the foot massage. Cycle through 3 massage intensities.
- STOP icon: Turn off the head/foot massage.

ONE TOUCH BUTTON

- Home icon: One touch flat position.
- Select icon: One touch Select button. Select head or foot zone.
- Anti-Snore icon: One touch **ANTI-SNORE** preset position.
- TV icon: One touch **TV** preset position.
- Z-G icon: One touch **Zero Gravity** preset position. **Zero Gravity** adjusts your legs to a higher level than your heart, helping to relieve pressure of the lower back and promote circulation.
- Sun icon: One touch underneath LED lighting on/off.

Pairing Remote

The original remote that comes in the box is already paired to the foundation. No further action is required. In the event when the remote is not paired with the foundation, follow the following to pairing instructions.

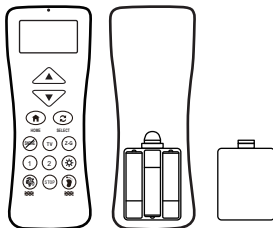

- STEP 1: Make sure the batteries are good. Replace them when needed.
- STEP 2: Unplug the power cord from the power source. If 9V backup batteries were installed in the power supply, remove them.
- STEP 3: Press and hold Select button (⌂) until the LCD screen of the remote control shows PAIR.
- STEP 4: Plug the power cord back into the power source. You will hear an activation sound in 2 seconds, indicating the remote is paired to the power foundation.

Setting Programmable Position

- STEP 1: Adjust the head, foot and neck to your desired position.
- STEP 2: Press and hold the program button (⌘) for about 5 seconds until the LCD screen shows **SAVE**, and press picture programmable buttons (1)(2) until you hear 2 beep sounds.
- STEP 3: To adjust a saved position, repeat step 1 and 2 and the new position will be saved.

Setting Remote Lock

LOCK

Press and hold the **HOME** (🏠) and **SELECT** (⌂) buttons simultaneously for about 5 seconds until the LCD screen shows **LOCK**.

UNLOCK

Press and hold the **HOME** (🏠) and **SELECT** (⌂) buttons for about 5 seconds until the **LOCK** disappears on the LCD screen.

LOW BATTERY


The LCD screen will show **LOW** once the batteries are in low power.

If simultaneous operation of two foundations is desired, please follow the steps listed below.

Note the sync cord is needed to control two foundations with mobile APP, follow the "Synchronizing Two Foundations with Sync Cord" instruction to install the sync cord or follow the "Wireless Syncing Instruction".


Wireless Syncing Instruction

- STEP 1: Make sure both remotes are individually paired to the foundation. If the remote is not paired with the foundation, follow the pairing instructions.
- STEP 2: Make sure both foundations are at home or flat position.
- STEP 3: Press and hold (⬆) and **HOME** (🏠) on both remotes at the same time for 5 seconds until the LCD screen shows **SYNC**.

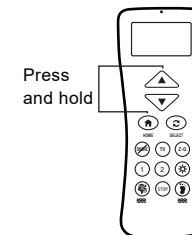


UN-SYNC Two Foundations

Press and hold (⬆) and (⬇) on both remotes at the same time for 5 seconds, until the LCD screen flashes and shows **UN-SYNC**.

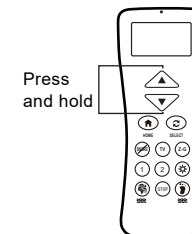

Synchronizing Two Foundations with Sync Cord

- STEP 1: Unplug foundations from the power source.
- STEP 2: Connect the male ends of the sync cord to the female sync plugs of the control boxes (Refer to below illustration).
- STEP 3: Plug foundations back into the power source.
- STEP 4: Check to ensure all cords are securely attached. Either remote will operate both foundations simultaneously.

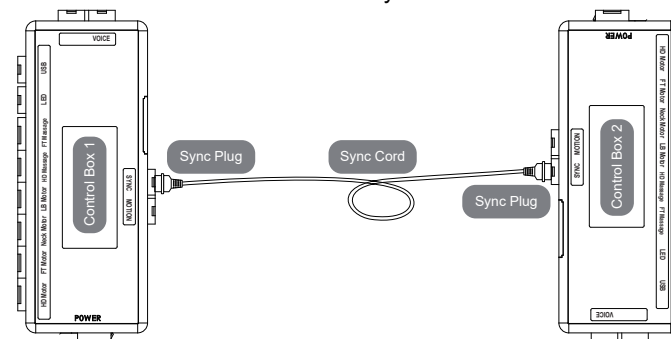

MotoSleep Mobile APP (Optional)

General Information

MotoSleep Mobile APP is a mobile application to control the MotoSleep Power Foundation with mobile devices. The application is available for both iOS and Android based smart devices.

The MotoSleep Mobile APP uses the latest technology to communicate with your MotoSleep Power Foundation. Make sure your mobile devices has the pairing function for the APP to work correctly.

Install the MotoSleep APP

IOS







- Open APP Store on your IOS device
- Tap the Magnify Glass Icon and search with the keyword **MOTOSLEEP**
- Tap on **GET** and **INSTALL** to install the APP
- Wait for the download/installation to be complete

Android



- Open www.motosleep.com from your browser.
- Scroll down to the bottom of the page and press download.
- In the download page, tap the download button to download the app.
- Open the download file and install the app.

The Built-in Owner's Manual at your fingertip

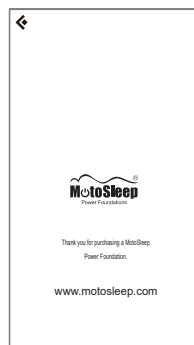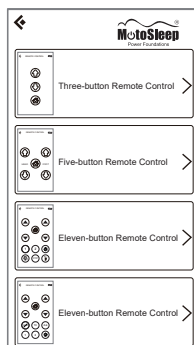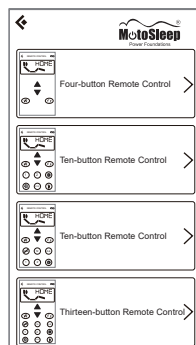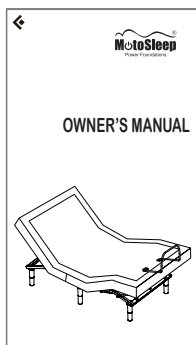


Device Pairing


Before you start. Switch on the pairing function on your mobile device. Make sure your MotoSleep Power Foundation is correctly connected and is power-on.

The MotoSleep APP will automatically pair your MotoSleep Power Foundation once you start the APP and does not require additional settings.

MotoSleep APP Home Screen

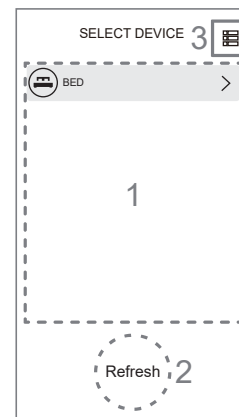

Home Screen

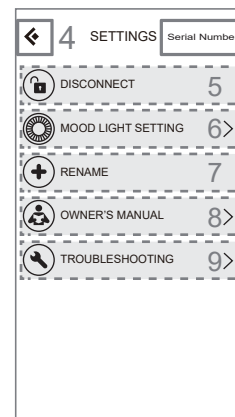


Settings Page

Home Screen	
1	List of MotoSleep Products within the connection range*
2	Tap the Refresh Button to refresh the MotoSleep Products List
3	Enter the Settings Page
Settings Page	
4	Back to the Home Screen
5	Disconnect from the paired MotoSleep Product
6	Choose the light colors, and set up the motion sensor timer
7	Rename your MotoSleep Power Foundation of your choice
8	Owner's Manual Page
9	Troubleshooting
10	Serial Number of the control box

* The effective connection range is $\pm 10m$

MotoSleep APP Remote Control

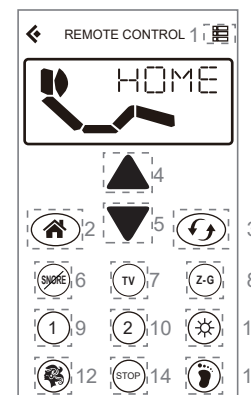

* Remote interface varies in between different MotoSleep Power Foundation models. If you wish to disconnect from the current paired Power Foundation, Enter the Settings Page and tap the Disconnect button.

Remote Control	
1	Enter the Settings Page
2	One touch flat position
3	Select Head, Foot, Neck or Tilt section
4	Lifting function
5	Lowering function
6	Anti-Snore Preset Position
7	TV Position
8	Zero Gravity Preset Position
9	Programmable Position 1
10	Programmable Position 2
11	Under Bed Lighting
12	Head Massage (3 different intensities)
13	Foot Massage (3 different intensities)
14	Massage Off Button

Product Serial Number

Each power foundation has its unique serial number. The serial number can be found on the metal frame near the legs at the foot area.

